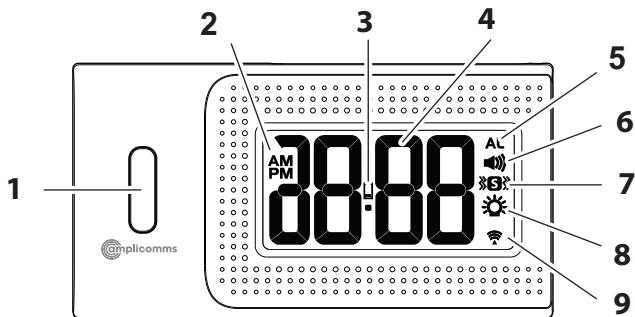
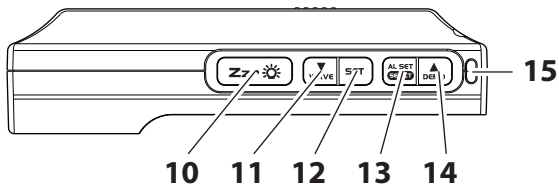
## **TCL 250**

**Bedienungsanleitung (DE)**  
**Operating Instructions (UK)**  
**Mode d'emploi (FR)**  
**Bedieningshandleiding (NL)**  
**Istruzioni d'uso (IT)**  
**Instrucciones de uso (ES)**  
**Bruksanvisning (SE)**  
**Manual de instruções (PT)**  
**Bruksanvisning (NO)**

# TCL 250

# TCL 250

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## Box contents

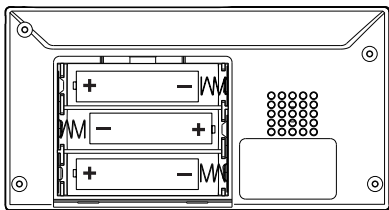
- Alarm clock
- Lanyard with clamp (for attachment eg to the pillow)
- (fabric) travel pouch
- User guide

## Insert batteries

To operate, your alarm clock requires 3 x AA alkaline batteries (not included).

1. Lay the device face down on a soft, flat surface.
2. Slide down the battery cover on the bottom of the device and remove it.
3. Insert 3 x AA alkaline batteries by observing the correct polarity marks "+" and "-" in the battery compartment.

4. Replace the battery cover.

Notes:

- Frequent use of the vibration alarm, strobe light / flashlight, and loud alarm will significantly affect battery life.
- The empty battery icon appears between the time digits when the battery is low.
- Dispose of used batteries according to the instructions.

## Environmental reception effects

Your radio controlled clock obtains the accurate time with wireless technology. Same as all wireless devices, the receiving ability maybe affected by, but not limited to, the following circumstances:

- Long transmitting distance
- Nearby mountains and valleys
- Among tall buildings
- Near railway, high voltage cable etc.
- Near freeway, airport, etc
- Near construction site
- Inside concrete buildings
- Near electrical appliances
- Near computers and TV's
- Inside moving vehicles
- Near metallic structures.

Place your clock at a location with optimal signal, i.e. close to a window and away from metal surfaces or electrical appliances.

**Note:** please keep your clock one meter / three feet away from any other electronics for best DCF / MSF reception.

## Radio controlled time signal reception

After your clock is powered up, it starts to receive radio controlled time signal. The  indicator flashes.

- Receiving signal ( indicator flashing)
- Successful reception ( indicator becomes static)
- Failed reception ( indicator disappear).

## Automatic reception and manual reception

### Automatic reception

Your clock starts automatic reception everyday at 1:00, 2:00, and 3:00. If it fails to receive the radio controlled time signal at 3:00, it will start reception at 4:00. If it fails again, it will start reception at 5:00. If it fails again, it will start automatic reception at 1:00 again in the next day.

During reception, all buttons do not function except the

 button.

### Manual reception

Simply press and hold **WAVE/▼** to enter manual setting mode. Your alarm clock will start manual reception with a short beep. To stop the function, press and hold the **WAVE/▼** button again.

## To set time and the snooze duration

(default time at 0:00 and 24Hr time format)

1. Press and hold down **SET**, "00" flash, press ▲/▼ to set the time zone (up to ±12 hours).  
"00" = GMT +1 hour (e.g. Germany)  
"01" = GMT +2 hours (e.g. Finland)  
"-01" = GMT 00 hour (e.g. U.K.)
2. Press **SET** again, the hour digits flash. Press ▲ or ▼ to set the time in hours (hold ▲/▼ to advance rapidly).
3. Press **SET** again, the minute digits "00" flash. Press ▲ or ▼ to set the time in minutes (hold ▲/▼ to advance rapidly).
4. Press **SET** again, the display shows time format and flashes. Press ▲ or ▼ to select "12HR" or "24Hr" time format.  
If "12Hr" is selected, the **AM/PM** indicator will appear.
5. Press **SET** again, the display shows "05" and flashes. Press ▲ or ▼ to select your snooze time from 5 to 60 minutes.
6. Press **SET** once again or no button around 10 seconds to exit the setting mode.

## To set the alarm

1. Press and hold **AL SET/SELECT**, the **AL** indicator appears and the alarm's hour digits flash. Press ▲ or ▼ to set the alarm in hours (hold ▲/▼ to advance rapidly).
2. Press **AL SET/SELECT** once again, the alarm's minute digits flash. Press ▲ or ▼ to set the alarm in minutes (hold ▲/▼ to advance rapidly).
3. Press **AL SET/SELECT** again, the «**S**» indicator flashes and the LCD displays "HI" (vibration at HI level). Press ▲ or ▼ to select the vibration level.
  - HI (strong vibration level)
  - LO (soft vibration level)
4. Press **AL SET/SELECT** once again, the  indicator flashes and the LCD displays "HI". Press ▲ or ▼ to select the alarm tone volume.
  - LO (soft)
  - HI (loud)
5. Press **AL SET/SELECT** once again or no button around 10 seconds to exit the alarm setting mode.

## To select the alarm mode

- At normal time display mode, press **AL SET/SELECT** one at a time to activate the alarm(s), the corresponding alarm indicator(s) in the display will be on:
  - »**S**»: only shaker alarm is activated
  - »**S**»**🔊**: both sound and shaker alarm are activated
  - »**S**»**☀️**: both strobe light and shaker alarm are activated
  - »**S**»**🔊**»**☀️**: strobe light, sound and shaker alarm are activated.
- Press **AL SET/SELECT** once more to turn off all the alarms, all »**S**», »**🔊**» and »**☀️**» alarm indicators disappear.

## To stop and reset the alarm

When alarm is sounding, the corresponding alarm indicator flashes. Press **AL SET/SELECT** once to stop the alarm and reset it to come on the following day. After that, the corresponding alarm indicator stays on the display.

## To use snooze

When alarm is sounding, press **Zz • ☀️** once, the alarm will be stopped temporarily and the corresponding alarm indicator keeps flashing. Alarm will come on again after the set snooze duration is over.

## To use the backlight

During the normal time display, briefly press  to turn on the display backlight for 5 seconds. This function is not available when the device is currently giving a wake-up signal.

## To use the demo mode

Press and hold the **▲/DEMO** button for 2 seconds to trigger the buzzer, stroke light and shaker.

## To use the flashlight

During normal time display, press and hold  to turn on the flashlight.

To turn it off, press and hold  again.

## Battery status indicator

When the battery level is low, the vibration level, alarm tone and brightness are reduced. The symbol  appears between the time digits. Replace the batteries with new AA alkaline batteries.

## Technical data / features

Functions: Clock, Alarm clock

Alarm modes: Buzzer, Vibration, Strobe light

Batteries (not included): 3 x AA

## Specifications

RCC Frequency	77.5 KHz
Alarm duration	1 minute
Snooze duration	5 to 60 minutes
Default settings:	
Time format	24 hour
Time	0:00
Alarm	6:00 (AM)
Alarm and shaker level	HI
Snooze duration	05 (5 minutes)

## Safety Information

- Read this manual carefully and keep it for future reference.
- Place your device on a stable surface, away from sources of direct sunlight or excessive heat or moisture.
- Protect the device from dripping or spraying water.
- Protect your furniture when placing your units on a natural wood and lacquered finish by using a cloth or protective material between it and the furniture.

## Disposal

You are obliged to dispose of consumable goods properly in accordance with the applicable legal regulations. The symbol on this product indicates that electrical and electronic apparatus must be disposed of separately from domestic waste at suitable collection points provided by the public waste authorities. Packaging materials must be disposed of according to local regulations.

## Maintenance

- Clean your unit with a soft cloth moistened only with mild soap and water. Stronger agents such as Benzene, thinner or similar materials can damage the surface of the unit.
- If the unit is not to be used for a prolonged period, such as a month or longer, remove the batteries to prevent possible corrosion. Should the battery compartment become corroded or dirty, clean the compartment thoroughly and replace the batteries.

## Troubleshooting

If your clock displays irrelevant time or does not function properly, possibly due to electrostatic charge or discharged batteries, remove the batteries of your alarm clock. Wait a few minutes, then reinsert the batteries or replace them with new batteries.

The alarm clock will reset to factory settings and you will need to reset it.

## Guarantee

AMPLICOMMS equipment is produced and tested according to the latest production methods. The implementation of carefully chosen materials and highly developed technologies ensure trouble free functioning and a long service life.

The period of guarantee is 24 months from the date of purchase. The terms of the guarantee do not apply to the rechargeable batteries or power packs used in the products. All deficiencies resulting from material or production faults which occur during the period of guarantee will be eliminated free of charge. Rights to claims under the terms of guarantee are annulled following intervention by the purchaser or third parties. Damage caused as a result of improper handling or operation, incorrect positioning or storing, improper connection or installation, Acts of God or other external influence are not covered by the terms of guarantee.

In the case of complaints, we reserve the right to repair or replace defect parts or provide a replacement device. Replacement parts or devices become our property. Rights to compensation in the case of damage are excluded where there is no evidence or intent or gross negligence by the manufacturer.

If your equipment shows signs of defect during the period of guarantee, please return to the sales outlet in which you purchased the AMPLICOMMS equipment together with the purchase receipt. All rights to claims under the terms of guarantee in accordance with this agreement must be asserted exclusively with regard to your sales outlet.

Two years after the purchase of our products, claims under the terms of guarantee can no longer be asserted.

## **Helpline**

If you need some help and you're in the UK, please call us on 0844 800 6503 (using an UK landline, at the time of going to print: calls cost 6p for connection charge and then 5p a minute. Mobile costs may vary). Lines are open Monday to Friday, 9am to 5pm, excluding public holidays. If you've a claim on your 2-year guarantee, please contact your supplier.

## Declaration of conformity

This device fulfils the requirements stipulated in the EU directive: 2014/53/EU.

Conformity with the above mentioned directive is confirmed by the CE symbol on the device.

To view the complete Declaration of Conformity, please refer to the free download available on our website

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Audioline GmbH